


Produktname	Einweg-KF94-Gesichtsmaske
Bfe.	BFE $\geq 95\%$
Material	Pp + schmelzen geblasen + schmelzen geblasen + pp
Farbe	Farbe abgestuft (anpassbar)
Stil	Gehörsschleife
Gewicht	50g + 25g + 25g + 30g oder als Ihre Anfrage
Größe	20x8cm für Erwachsene, Ansprechpartner für Kindergröße
Verpackung	50 teile / sack, 40 taschen / karton, 2000 stücke / karton
Lieferzeit	7-15 Tage
Zahlungsbedingungen	T / t, l / c, paypal usw
Bedienung	OEM, Proben bereitgestellt, Erbringung der Produktspezifikation
Prüfzertifizierung	GB2626-2019.

Proben können wie gewünscht bereitgestellt werden. Willkommen Ihre Anfrage.

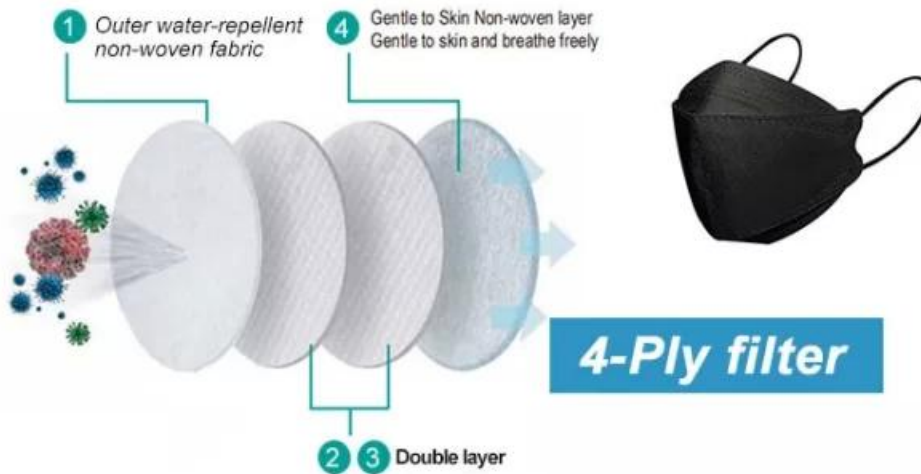

# HOW TO USE



- 1 Remove mask from packaging.
- 2 Wear the mask to cover your nose and chin.
- 3 Hook each strap over your ear to lock them in place
- 4 Press the nose piece with the fingers of both hands so that the mask is in close contact with your face.

## Precautions for Use

1. Make sure not to place towel, tissue, etc. inside of the mask.
2. Do not use if the inside of the mask is contaminated.
3. Do not touch the surface of the mask after wearing.
4. This mask is disposable. Do not wash your mask.