


nome del prodotto	Disposable coverall		
Intended use	This disposable coverall is designed for Hospital, Household, Emergency, Car industry, Waste management, Gardening, Pharmaceutical, Food processing, Painting, Outing, biological chemical hazard, lab, rescue and relief, Mining, Oil and gas		
Istruzioni per l'uso	<p>Donning</p> <p>1) Choose your correct size. Ensure that the product is suitable for the application.</p> <p>2) take out the coverall, Unfold the coverall carefully Check your coverall for any defects such as holes or tears and replace if damaged.</p> <p>3) Spostarsi in un'area lontana dai rischi di contaminazione e sul posto di lavoro. Assicurati di avere tutto il tuo equipaggiamento protettivo personale facilmente raggiungibile. In una posizione seduta, rimuovere calzature, abbigliamento ingombrante, gioielli e qualsiasi altra cosa che possa interferire con il proprio DPI o posare un rischio di sicurezza.</p> <p>4) mettendo il tuta Senza calzature, inserisci con cautela la gamba nella tuta, puntando il piede e guidandolo con cautela attraverso l'apertura della caviglia. Ripeti con un'altra gamba e poi tira la tuta in vita. Inserisci le braccia alle maniche.</p> <p>5) Arrotolare le gambe della tuta e mettono le calzature, quindi rotola le gambe della copertura verso il basso per coprire completamente la gamba inferiore e la caviglia Tirare il cappuccio sopra la testa, tirare lentamente la zip dal basso verso l'alto e fino al mento.</p> <p>Amoroso</p> <p>1) Rimuovi il maggior numero possibile di contaminazione dai tuoi tute.</p> <p>2) Spostarsi in un'area designata lontano dai rischi di contaminazione e sul posto di lavoro. Assicurarsi di essere facilmente raggiungibile dalle strutture di smaltimento appropriate. Assicurati di avere qualcuno o qualcosa da tenere per stabilità.</p> <p>3) A partire dal cappuccio, tira giù il rivestimento e uscire da esso. removing the top part of the body and arms by rolling it down and inverting the sleeves. roll down the coverall and step out.</p>		
codice	OH-5000	dimensione	L
modello	Coverall with hood, double-sided tape over the entire flap. Elastico hood and ankles, cut and sewn seams, Colore bianco		
Materiale	60% Polypropylene + 40% Polyethylene, 55 ±3% gsm,		
Certificato	Type5 6, EN ISO 13688:2013 ,EN ISO 13982-1:2004 +A1:2010, EN 13034:2005 +A1:2009 ,EN 14325:2004 FDA, ISO12385		
Imballaggio	10. pc s in un poli sacchetto, 50. p cs / cartone, Dimensione cartone: 50 * 30 * 50 cm Caricamento Qtà: 18000pcs in 1 * 20gp, 45000pcs in 1 * 40HQ		
Data di scadenza	3 anni		

Technical specification

(All measurement are for the coverall)

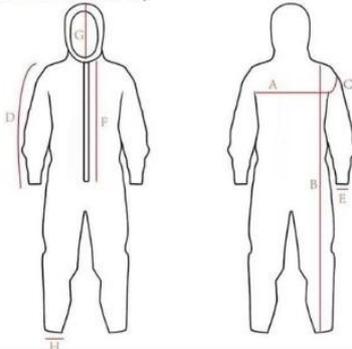

	No.	S	M	L	XL	XX L
chest	A	110	120	130	140	150
Total length	B	160	165	170	175	180
Length of sleeve	D	58	58	60	60	60
Length of hood	G	18	18	18	18	18

orient Honest
MEDICAL INTERNATIONAL



CE ISO FDA
ISO 13485

orient Honest
MEDICAL INTERNATIONAL



CE ISO FDA
ISO 13485

 orient Honest
MEDICAL SUPPLY CO., LTD.



CE  **ISO 13485** **FDA**