



□□□	□□□ KF94 □□ □□□
BFE.	BFE≥95 %
□□	PP + Melt Blown + Melt Blown + PP.
□□	□□ □□ (□□□□)
□□□□	□□ □□
□□	50g + 25g + 25g + 30g, □□ □□□ □□□□
□□	□□□□□ 20x8cm, □□□□ □□□ □□□□□□
□□	50pcs / □□, 40bags / carton, 2000pcs / carton
□□ □□	7-15 □
□□ □□	T / T, L / C, □□□ □
□□□□	OEM, □□□ □□, □□ □□ □□
□□ □□□	GB2626-2019.

□□□ □□ □□□ □□ □ □ □□□□. □□□□ □□□□ □□□□□□.

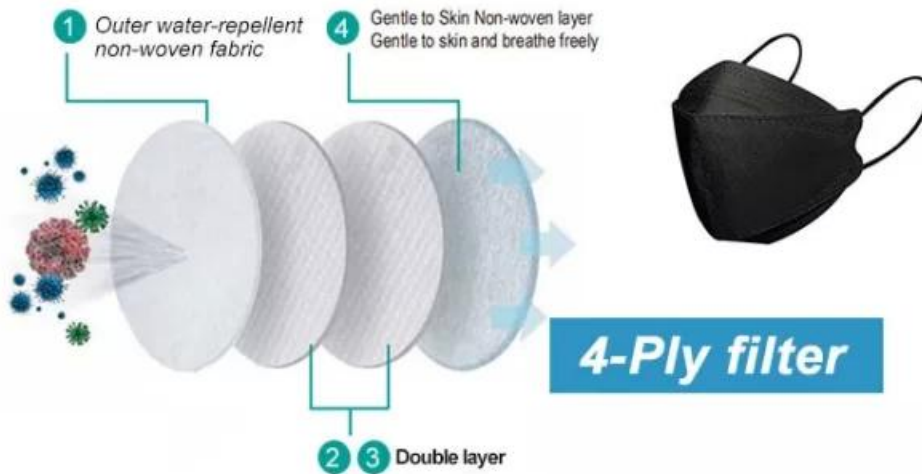

	<p>Invisible Adjustable Soft Nose Clip Better Fit Your Face Barrier the Up Direction Air</p>		<p>High Strength Ear Loop Comfortable Ware No Hurt or Harm to Your Ear with Long Period Wearing</p>		<p>Extra Soft Accessible Skin Layer inside soft Non-Woven Protective Layer No Harm to your Skin</p>
--	------------------------------------------------------------------------------------------------------	--	-----------------------------------------------------------------------------------------------------------------	--	-----------------------------------------------------------------------------------------------------------------

# HOW TO USE

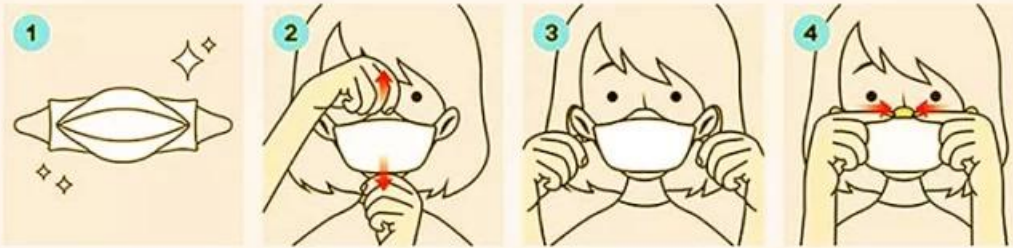

- 1 Remove mask from packaging.
- 2 Wear the mask to cover your nose and chin.
- 3 Hook each strap over your ear to lock them in place
- 4 Press the nose piece with the fingers of both hands so that the mask is in close contact with your face.

## Precautions for Use

1. Make sure not to place towel, tissue, etc. inside of the mask.
2. Do not use if the inside of the mask is contaminated.
3. Do not touch the surface of the mask after wearing.
4. This mask is disposable. Do not wash your mask.