



□□□	3D □□□ □□□ KF94 □□ □□□□□□
BFE, PFE.	BFE≥95 %
□□	PP + Melt Blown + Melt Blown + PP.
□□	□□ □□ □□ □□ □, □□ □□□□ □□
□□□	□□ □□
□□	50g + 25g + 25g + 30g □□ □□□□ □□
□□	□□□□□□ 20x8cm, □□□□ □□□□ □□
□□	50pcs / □□, 40bags / carton, 2000pcs / carton.
□□ □□	7-15 □
□□ □□	T / T, L / C, X- □□, PayPal, □
□□□	OEM, □□ □□, □□ □□ □□
□□ □□□	GB2626-2019.

□□□ □□ □□□ □□ □ □ □□□□. □□□□ □□□□□□.

- * □ □□ □□□ □□□□ 4 □□ □□□□
- * 3D □□□ □ □□□□□ □ □□ □□ □□ □ fresher □□
- * □□□□ □□, □□□ □□□□□□ □□□ □□□□□□ □□□
- * □□□□ □□ □□□□
- * □□ □ □□□□□ □ - □□□



□□ □□ □□ □□ □□, □□, □□ □□ □□

HOW TO USE



- 1 Remove mask from packaging.
- 2 Wear the mask to cover your nose and chin.
- 3 Hook each strap over your ear to lock them in place
- 4 Press the nose piece with the fingers of both hands so that the mask is in close contact with your face.

Precautions for Use

1. Make sure not to place towel, tissue, etc. inside of the mask.
2. Do not use if the inside of the mask is contaminated.
3. Do not touch the surface of the mask after wearing.
4. This mask is disposable. Do not wash your mask.



Company Profile



Factory outside



Material warehouse



Dust free mask workshop



Dust free sewing workshop



Dust free packing workshop



Finished products warehouse



CE



ISO 13486



FDA