


Product Description

| | |
|-----------------|---|
| Name | Disposable KF94 face mask |
| Material | Nonwoven face mask |
| Color | Black,Blue , Red, Green, Yellow |
| Place of Origin | Hubei, China (Mainland) |
| Specifications | 1.4-ply protection,highly breathable 2.PFE>95%,BFE>95% 3.Style: Adjustable nose bridge Tie on 4.high filtration efficiency |
| Ply Type | 4 ply |
| Style | earloop |
| Applications | hospital,laboratory,food industry,dust free workshop,electronic manufacturers and so on |
| Packing | 20pcs/box,2000pcs/ctn,or as your requirements |
| Note | 1.This product is sterilized by ethylene oxid 2.Disposable |
| Remark | all can be special customized according to your actual |

HOW TO USE

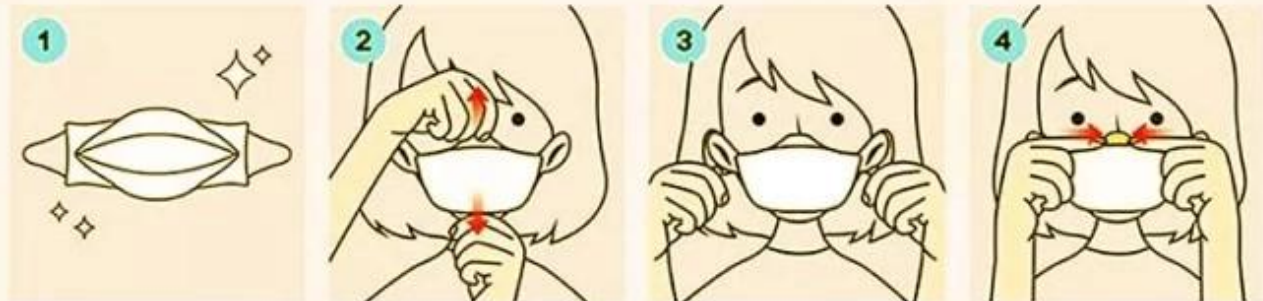

- 1 Remove mask from packaging.
- 2 Wear the mask to cover your nose and chin.
- 3 Hook each strap over your ear to lock them in place
- 4 Press the nose piece with the fingers of both hands so that the mask is in close contact with your face.

Precautions for Use

1. Make sure not to place towel, tissue, etc. inside of the mask.
2. Do not use if the inside of the mask is contaminated.
3. Do not touch the surface of the mask after wearing.
4. This mask is disposable. Do not wash your mask.