


product name	Disposable Coverall		
Intended use	This disposable cover is designed for hospitals, households, emergencies, car industries, waste management, gardening, pharmacy, food processing, paintings, sightseeing, biological chemical hazards, laboratories, rescue and assistance, mining, oil and gas		
Instructions for use	<p>Donning</p> <p>1) Select your correct size. Make sure the product is suitable for applications.</p> <p>2) Remove the coverall, open the coverall carefully. Check your cover for all defects such as holes or tears and replace if damaged.</p> <p>3) Move to an area far from contamination and hazards at work. Make sure you have all your personal protective equipment easily accessible. In a sitting position, remove footwear, thick clothes, jewelry and anything that can be annoying AP you or pose a safety risk.</p> <p>4) Placing a cover without footwear, put your feet carefully into the coverall by pointing your feet and carefully guiding it through the opening of the ankle. Repeat with the other legs and then pull the cover to your waist. Put your arms into the arm.</p> <p>5) Roll foot cover and place your footwear, then roll the foot down to fully cover the lower leg and ankle.</p> <p>method to the chin.</p> <p>Doffing</p> <p>1) Remove as many contamination as possible from your cover.</p> <p>2) Move to the designated area far from contamination and dangers at work. Make sure you are easily accessible from the appropriate disposal facility. Make sure you have someone or something to hold stability.</p> <p>3) Starting from the hood, pull the cover down and out of it. Remove the top of the body and arms by rolling it and turning the sleeve of the shirt. Roll down Coverall and come out.</p>		
code	OH-5000	size	L
model	Coverall with a hood, a two -sided tape throughout the flap. Elastic hood and ankle, cut and stitches sewn, white		
Ingredient	60% Polypropylene + 40% Polyethylene, 55 ± 3% GSM,		
Certificate	Type5/6, EN ISO 13688: 2013, EN ISO 13982-1: 2004 + A1: 2010, En 13034: 2005 + A1: 2009, en 14325: 2004 FDA, ISO12385		
Is packing	10 pcs in poly bags, 50 pcs/carton, Cardboard Size: 50 *30 *50cm Load Qty: 18000pcs in 1*20gp, 45000 pcs in 1*40HQ		
Shelf life	3 years		

		Technical specification					
(All measurement are for the coverall)		No.	S	M	L	XL	XX L
	chest	A	110	120	130	140	150
	Total length	B	160	165	170	175	180
	Length of sleeve	D	58	58	60	60	60
	Length of hood	G	18	18	18	18	18

Taking pictures:

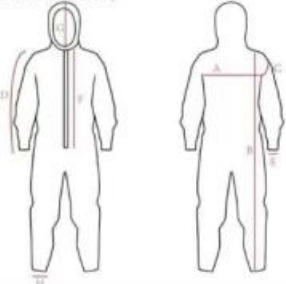

orient Honest  
MEDICAL INTERNATIONAL



orient Honest  
MEDICAL INTERNATIONAL

