

CONFIGURATION OVERVIEW

This [gown](#) is designed for optional protection during surgical procedures of moderate duration involving

moderate risk of infection.

. It consists of:

Material: SMS,PP,PP+PE,CPE

Color: blue, white

2 long sleeves with knitted cuff or elastic cuff

Velcro back neck, tie on neck or round collar and waist

TECHNICAL SPECIFICATIONS

Recommend Raw Material: [SMS](#), PP+PE, CPE

Style: Two long sleeves with knitted cuff, velcro back neck or tie on neck and waists

Dimensions: cm

Sizes	Long (below highest point of shoulder seam)	Width (monolayer)	Belt length	Length of sleeve	Deviation
108×138 (S)	108	137	65	58	±2
115×137 (M)	115	138	65	58	±2
120×150 (L)	120	150	65	59	±2
130×150 (XL)	130	150	69	59	±2
140×160 (XXL)	140	160	69	61	±2

Packaging

1pc/bag, 50pcs/Carton

Carton: 50cm*32cm*30cm, Around 540 cartons in a 20GP, 1080 cartons in one 40HQ

INSTRUCTIONS

1. Washing hands clean ;
2. Checking the sterile surgical package(not expire,damaged,wet and so on..)
3. Take off the sterile bag and take out the [gown](#) and open it.

4. Catching the gowns collar, tie on neck side should be face to the body, slight shake the gowns to make sure it is totally opening;
5. Putting your hands into according two sleeves, until your hands totally come out;
6. Fixed tie on neck and waist ,adjust the ties fit nicely on the body, conforming to the contour of your body.

Stitched seams




Ultrasonic sealing seams





orient Honest
MEDICAL INTERNATIONAL

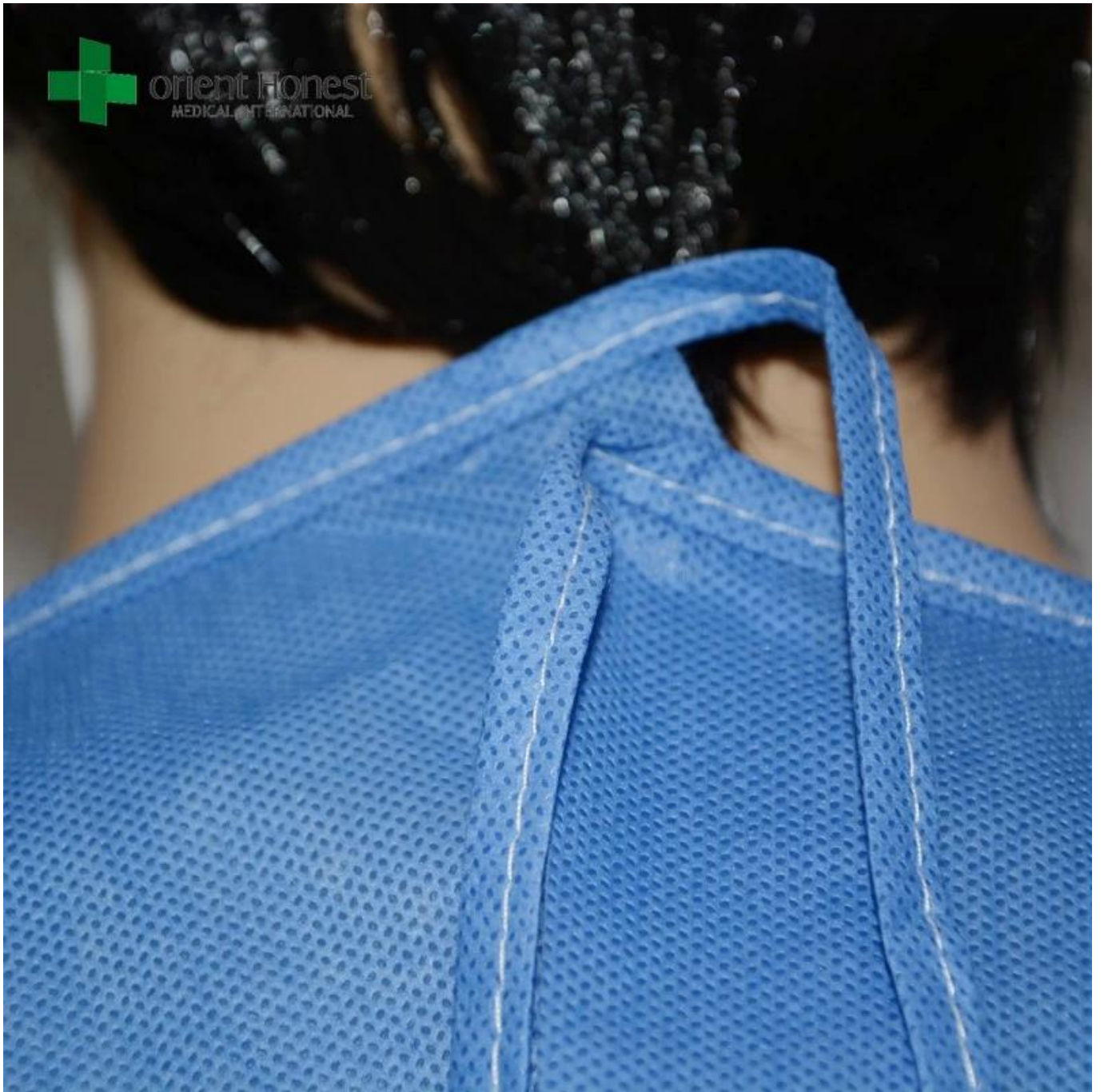




orient Honest
MEDICAL INTERNATIONAL





orient Honest
Klasifikasi: 100% Polyester

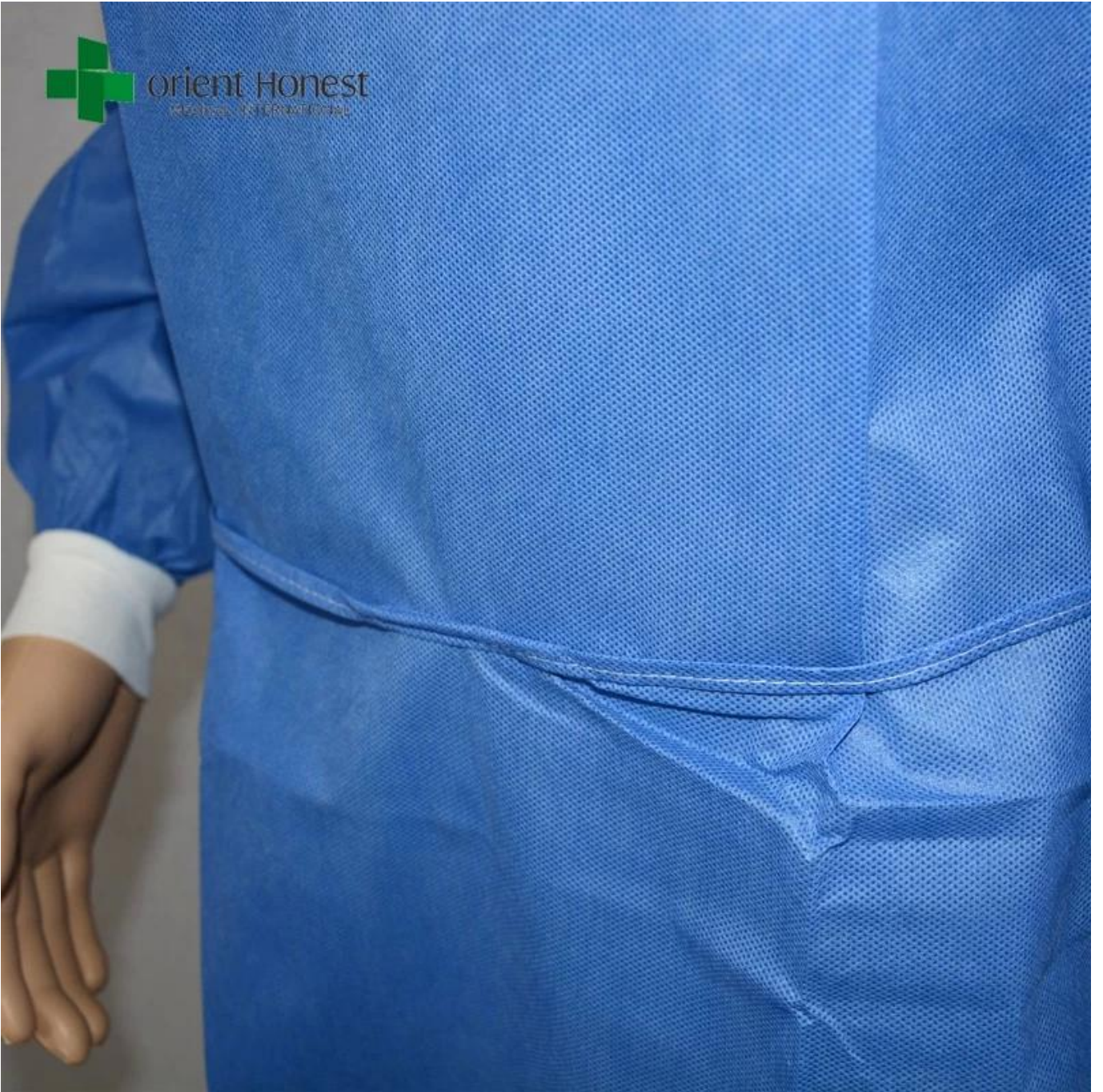



orient Honest
MEDICAL INTERNATIONAL





orientHonest
MEDICAL INTERNATIONAL

