CONFIGURATION OVERVIEW

This <u>gown</u> is designed for optional protection during surgical procedures of moderate duration involving

moderate risk of infection.

. It consists of:

Material: SMS,PP,PP+PE,CPE

Color: blue, white

2 long sleeves with knitted cuff or elastic cuff

Velcro back neck, tie on neck or round collar and waist

TECHNICAL SPECIFICATIONS

Recommend Raw Material: <u>SMS</u>, PP+PE, CPE

Style: Two long sleeves with knitted cuff, velcro back neck or tie on neck and waists

Dimensions: cm

Sizes	Long (below highest point of shoulder seam)	Width (monolayer)	Belt length	Length of sleeve	Deviation
108×138 (S)	108	137	65	58	±2
115×137 (M)	115	138	65	58	±2
120×150 (L)	120	150	65	59	±2
130×150 (XL)	130	150	69	59	±2
140×160 (XXL)	140	160	69	61	±2

Packaging

1pc/bag, 50pcs/Carton

Carton: 50cm*32cm*30cm, Around 540 cartons in a 20GP, 1080 cartons in one 40HQ

INSTRUCTIONS

- 1. Washing hands clean;
- 2. Checking the sterile surgical package(not expire,damaged,wet and so on..)
- 3. Take off the sterile bag and take out the gown and open it.

- 4. Catching the gowns collar, tie on neck side should be face to the body, slight shake the gowns to make sure it is totally opening;
- 5. Putting your hands into according two sleeves, until your hands totally come out;
- 6. Fixed tie on neck and waist ,adjust the ties fit nicely on the body, conforming to the contour of your body.











